ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INDICATIONS: The DonJoy Adjustable OA, Force ACL, PCL, CI Knee Brace is designed to provide support to the knee. It may be suitable for those suffering from uni-compartmental knee osteoarthritis Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: None

APPLICATION INSTRUCTIONS

- 1) While sitting on the edge of a chair, step through the brace. Flexing the knee to 45°, position the brace as shown in figure 1, centering the hinge of the brace over the back of the leg.
- 2) Secure the bottom cuff strap first (#1), then the top high strap (#2).
- 3) Keeping the knee bent at 45°, push both condyles (hinges) toward the back of the leg, as shown in figure 2.
- 4) Maintaining the position of the hinges; for ACL version secure lower thigh strap (#3), for PCL version secure upper cuff strap (#3); for CI version secure lower high strap (#4) then upper cuff strap (#4) on the back of the leg.
- 5) For ACL version, draw the hinges forward slightly before fastening the front tibia (shin) strap. For PCL version draw the hinges forward slightly before fastening the front high strap (#4). For CI version draw the hinge strap forward slightly. Do not pull the hinge strap tight in the middle of the leg. F-For ACL version next, secure the upper cuff strap (#4). Be careful not to pull this strap too tight and risk circulation problems.
- 6) G-When all strapping is complete, the hinge should center just above the top of the knee and slightly behind midline. To ensure that no slippage occurs during activity, it is recommended to do a few knee bends or walk for a few minutes, then re-tighten the straps following steps B-F or B-E for CI version.

INSTRUCTIONS FOR ADJUSTABLE LOADING HINGE

Your brace frame has a base corrective load built into it. You may "fine tune" your base load by doing the following: Turn the adjustment screws clockwise until the enclosed Allen wrench tool. You may vary the front and back screw adjustment depending on its interface with the knee and comfort of load.

DO NOT turn any 1 screw more than 4 revolutions without toggling back to the other adjustable screw. Be careful not to bind the adjustment mechanism. If you exceed 10 full revolutions or the patella may become dislodged. If you exceed 10 full revolutions or the patella may become dislodged. Always load brace according to your comfort and activity level.

NOTE: Even if you don't feel immediate relief, it doesn't mean the brace isn't working. It can take several weeks for soft tissue in the knee to adjust to the increased load provided by the brace.

INSTRUCTIONS FOR CHANGEABLE STOPS

Step 1: Remove the 10° extension stops installed. To change the installed stops, first completely remove the screw as shown. Then replace the stops. Insert the new stops, align with the holes in the hinge plate, then reinset the screw. Using a Phillips head screwdriver, ensure that the screws are seated in the recess and completely tightened. (Fig. 5)

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• Hand wash your liners and condyle cups in cold water with mild detergent. AIR DRY ONLY. DO NOT HEAT DRY LINERS OR PADS.

• The hinge mechanism requires little maintenance in normal use. However, you may apply a dry lubricant such as a Teflon® spray to the internal mechanism of the hinge. Dry lubrication is recommended following use.

• Periodically inspect hinge screws for tightness and retighten as needed.

WARNINGS AND PRECAUTIONS:
This product is to be prescribed and fitted by a healthcare professional. The frequency, duration of use and directions for use should be determined by a healthcare professional. Discomfort, pain, swelling, changes in sensation or other unusual reactions occur while using this product, should you contact your doctor immediately. If you develop an allergic reaction and/or experience itch, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately. Do not use open wounds. Do not use the device if it is damaged and/or the packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

NOTA: NOT MADE WITH NATURAL RUBBER LATEX.

For single patient use only.
RX ONLY

WARRANTY: The DonJoy Brace comes with a 6 month warranty against material and workmanship on all softgoods with the brace. The DonJoy® brace has a lifetime warranty on the frame and hinges for braces with a Fourcepoint or steel gear hinge. Braces without a Fourcepoint hinge have a 5 year warranty.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. IT IS THE RESPONSIBILITY OF THE PATIENT TO NOT USE THIS PRODUCT IF THERE IS A MEDICAL CONTRAINDICATION. NO WARRANTY OR GUARANTEE OF USE WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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INDICACIONES: The DonJoy Adjustable OA, Force ACL, PCL, CI Knee Brace is designed to provide support to the knee. It may be suitable for those suffering from uni-compartmental knee osteoarthritis Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.

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DO NOT turn any 1 screw more than 4 revolutions without toggling back to the other adjustable screw. Be careful not to bind the adjustment mechanism. If you exceed 10 full revolutions or the patella may become dislodged. Always load brace according to your comfort and activity level.

NOTE: Even if you don't feel immediate relief, it doesn't mean the brace isn't working. It can take several weeks for soft tissue in the knee to adjust to the increased load provided by the brace.

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