

# CalorDerm Knee Compression & Heat Massager - Cordless



User Manual

## Introduction

---

Thank you for using the intelligent knee massager. This product allows you to perform knee air pressure massage, hot compress and vibration anytime and anywhere according to your own needs, and relieve knee pressure and fatigue.

Thank you sincerely for  
choosing this product

Please read this manual carefully before use  
Please keep it safe for future inquiry

# Product introduction —

---

## Packing list

1. Device x1
2. TYPE-C charging line x 1
3. Manual ×1


## Parameter

- Input voltage: 5V==1A
- Operating voltage: 3.7V
- Battery capacity: 2500mAH
- Operation time: 15 minutes

# Product introduction—

## Introduction to operation panel

- Visual LED large screen display, physical buttons, clear functions and simple operation
- 3D wrap-around three-stage full wrap, which fits knee joint and leg
- Three massage models, contact air pressure massage, segmented massage, balanced stress
- Three heating levels: Low-45 degrees; Middle-50 degrees; High-55 degrees
- High frequency micro vibration
- 15-minute regular massage

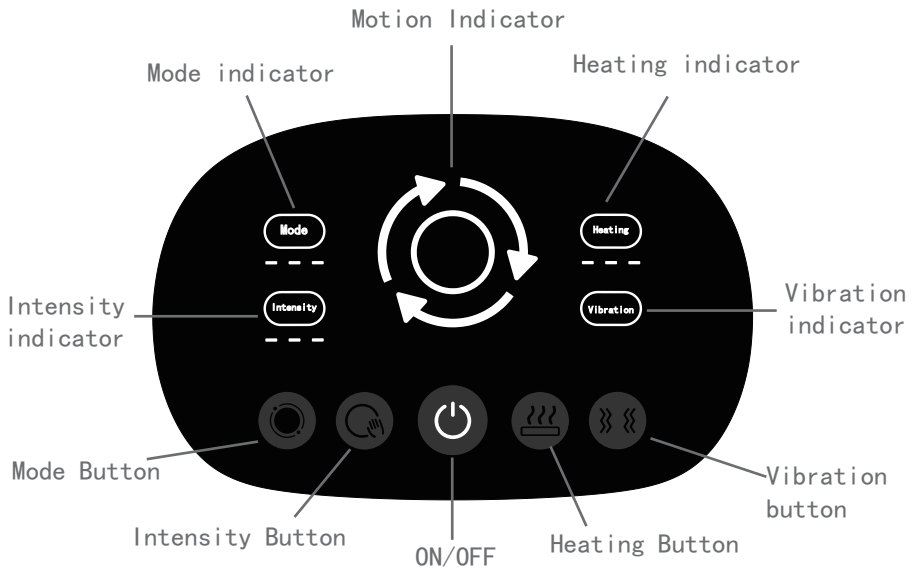

## Product introduction —

---

### How To Use



1. Wear the product on the knees and legs, fix and tie the product tightly with hook and loop tape, and fit the knees and legs.
2. Press the switch key for a long time to start the machine. After starting the machine, the default mode is one, the first gear of strength and the first gear of heating.
3. Press the mode key briefly to switch modes.
4. Press the strength key briefly to adjust the massage strength.
5. Press the heating key briefly to adjust the heating temperature (low-medium-high-turn off the heating cycle in turn).
6. Press the vibration key briefly to turn on and off the vibration.

## Points to note —

---

### Charging

1. Before the first use, please fully charge the product and charge it for about 3.5 hours.
2. It is not recommended to completely drain the battery.
3. Please charge in time when you are prompted for low power.
4. When charging, connect the plug wire of the Type-c interface to the charging port of the product.
5. The USB interface at the other end is plugged into the USB charging head and connected to the power supply.

### Battery

- Because the electromagnetic field generated by the motor of the massager is relatively small, it will generally not affect the electronic equipment and human body!
- The device (including lithium battery) and accessories of this product have reached the service life, so please do not throw them away at will, otherwise, e-waste will pollute the environment. Please make corresponding treatment according to the relevant regulations of the local environmental protection department on this kind of e-waste.

## Caution —

---

### **Notification**

- It is forbidden to stuff sundries into the gap of massage instrument.
- Do not pour liquid on the product, so as to avoid accidents caused by short circuit.
- Do not unplug the charging cable with wet hands.
- Do not use accessories and accessories that are not recommended.
- Do not use this product outdoors or in high temperature and humidity.
- Do not take part in any strenuous exercise after wearing the product.
- It is forbidden to lie or sit on the product.
- It is forbidden to bring huge impact and load to the product.
- Do not use it while sleeping.

## Safety precautions for use —

---

### Useful tips

Before use, you must know the following, otherwise it may cause damage or danger!

- There are leg nerve disorders or skin sensory disorders.
- People who inject metal, silicone or plastic into their knees due to fracture or surgery.
- Implantable medical electronic instruments that are easily affected by electromagnetic faults such as pacemakers.
- Those with malignant tumors.
- People with heart disease.
- A person who is unstable in early pregnancy or soon after giving birth.
- Diabetes causes perceptual disorders.
- Osteoporosis, severe fracture, sprain, muscle pain and acute pain should be guided by doctors.
- This product is not a medical device and is not recommended for medical use.
- This product contains a lithium battery, so please do not disassemble, modify, immerse in water or throw it into the fire, otherwise you can cause product damage.

### Safety precautions for use

In case of the following conditions, please shut down the machine immediately and contact the designated service center of the machine.

- If severe pain occurs during massage, please stop massage immediately and seek medical advice.
- If you notice any mechanical failure or abnormal condition in use, please stop using it immediately.



## Product maintenance—

---

### Maintenance, storage and transportation of instruments

#### ●Cleaning:

Wipe with a soft cloth, do not wet the host, and do not wipe the host with a wet towel to avoid water seepage. Do not wipe with thinner, gasoline or other flexible liquid when entering the host.

After stopping use, press the switch button for a long time to turn off the machine, and use the plastic bag for storage only after the instrument has normal heat dissipation.

#### ●Storage, custody:

Keep the instrument out of reach of children, away from humidity, high temperature, direct sunlight and water splash.

Do not disassemble, repair or replace it without authorization; Parts, which may cause accidents and failures.

If the equipment is not used for a long time, the product should be charged once a month to prevent the battery life from being shortened.

#### ●Transportation:

The product is designed with simple shockproof facilities, so please avoid large amplitude shaking during transportation.

No throwing, falling, rain, snow and exposure.

## Trouble shooting method —

Usage failure	Reasons	Solution
The product does not work properly	The product has just been completed for 15 minutes	Please press the power switch again to restart
Product is not heated	When used outside the recommended ambient temperature range, the heating effect is not obvious	It is recommended to use at the ambient temperature of 15-35 degrees Celsius
Abnormal sound of products	This sound is caused by the mechanical structure of the product	This sound is normal, which does not mean that the product function is faulty
Stop during operation	The battery is exhausted	Connect the charging cable to charge
	After 15 minutes, the massage will stop automatically	restart

### Transportation environment

- Temperature:  $-20^{\circ}\text{C}\sim+50^{\circ}\text{C}$
- Relative humidity:  $\leq 90\%$
- Atmospheric pressure:  $70\text{kPa}\sim 106\text{kPa}$

### Working environment

- Temperature:  $+5^{\circ}\text{C}\sim+40^{\circ}\text{C}$
- Relative humidity:  $\leq 80\%$
- Atmospheric pressure:  $86\text{kPa}\sim 106\text{kPa}$