

# DEEP TISSUE MASSAGER

...with easy to use heat and speed controls!



Neck & Shoulder

## IMPORTANT SAFETY INSTRUCTIONS

The following basic safety guidance must always be followed when using electrical appliances to reduce the risk of fire, electric shock and/or injury to any person:

### Warning

1. Read all instructions fully & Keep all instructions for future reference.
2. Not suitable for children
3. Do not use while driving
4. Do not use if you are pregnant
5. Do not use if you have any of the following health conditions: Heat sensitivity, Circulation disorder, varicose vein, bruises, broken skin, skin irritation, muscle or tissue inflammation, vein inflammation, injury, or any open wounds.
6. If you experience any pain or discomfort during the massage please stop immediately and consult your doctor
7. Do not use if you have any of the following: Pacemaker, artificial joints, electronic implants.
8. Do not use this product if you have applied any oils or lotions to your skin
9. Power off and disconnect product from the mains after use
10. Take precautions to make sure you do not fall asleep during use
11. Do not carry this product by the power cord
12. Never clean this product unless it is unplugged. Use a cloth dampened with a very small amount of water to clean. can be used to wipe the product but it must be dried immediately. Do not put this product back in until it is totally dry.
13. Do not operate under a blanket or pillow. Excessive heating can occur or cause fire, electric shock or injury to persons.
14. Never use this product anywhere near water or high humidity. Never use this product unless you are totally dry.
15. Do not immerse or expose this product or any other liquid for any reason.
16. This appliance is suitable for personal household use only. Commercial or professional use will void the warranty. The supplier cannot be held responsible for injury or damage if the appliance has been used for anything other than its intended use.
17. This appliance must be kept out of the reach of children.
18. Always disconnect this appliance from the power supply when unattended
19. Do not leave this appliance unattended while plugged in
20. Do not allow any leads or cords to rest in any position that might cause any trip or pull hazards.
21. Do not allow the power cord near any heated surfaces
22. Do not use the appliance if it has been dropped or damaged in any way.
23. Check the power cord and plug regularly for any damage. If there is any damage to the power lead or plug it must be repaired by the manufacturer or a qualified professional.
24. Use only original parts if this appliance needs repair or maintenance.
25. If the appliance begins to malfunction in any way you must unplug and stop using it immediately.
26. Never try to inspect or open the product in any way
27. Never operate or handle the appliance with wet hands.
28. Avoid using before bed – the massage can have a stimulating effect and may delay your sleep.
29. Keep all packaging provided with this appliance out of the reach of children.


## Usage Guide

Vary pressure according to your needs – if the kneading is too rough then remove some pressure


### Massage Areas:




Neck&Shoulder

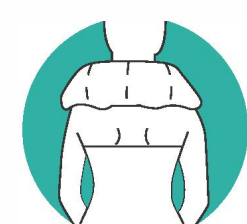


Neck

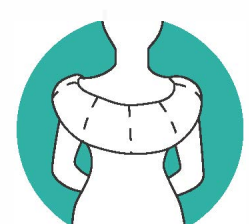

Back

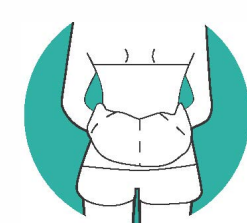


Waist

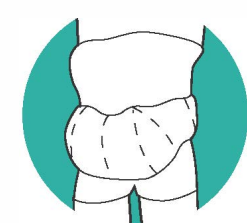


Abdomen



Thighs

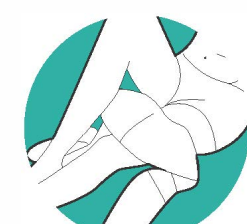

Legs

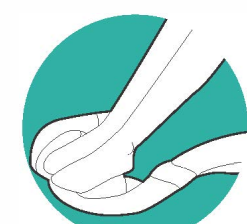


Foot




Calf

## Parts List

### A Main Unit

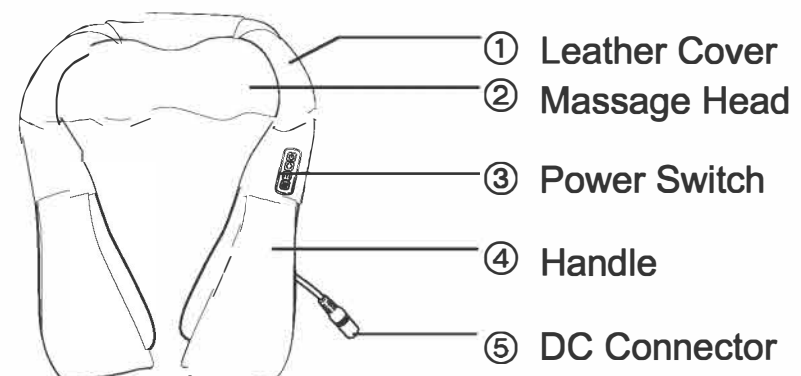

### B Power Adaptor

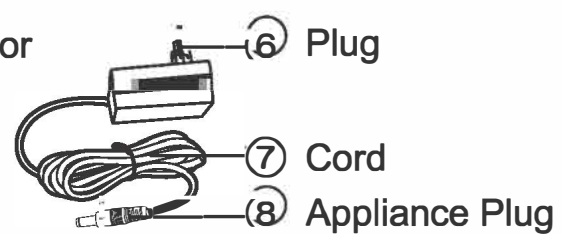

### C Vehicular Cigar-Lighter



## Technical Data

Rated voltage: Dc12v  
Rated Power: 24W  
Rated Frequency: 50HZ

Product features or design might change without notice.

## Function Guide

	<b>On/Off</b> Quick and easy to turn on and off. Automatic timer turns off after 15 minutes.
	<b>Speed</b> Easily adjust speed of kneading according to your requirements.
	<b>Direction</b> Easily adjust the direction of the kneading movement to target key areas of tension.
	<b>Heat Mode</b> Easily toggle heat function on and off.

## Troubleshooting

Problem	Solution
Unable to start the massager	Make sure the power plug is in the socket. Confirm the switch is on. Confirm if the button has been pressed.
Suddenly stops working during use	The massager will automatically finish after 15 minutes. Check if the power plug has pulled out. Check for any bad connections and cease use immediately if any are found. If none of the above work then turn off the main power supply and wait for one hour so that the product can cool down. The massager has an automatic overheating protection feature. If the problem persists stop using the product immediately and contact your retailer for advice.