




Spine Bracing System

CLINICIAN APPLICATION INSTRUCTIONS

EXOS FORM™ II 621

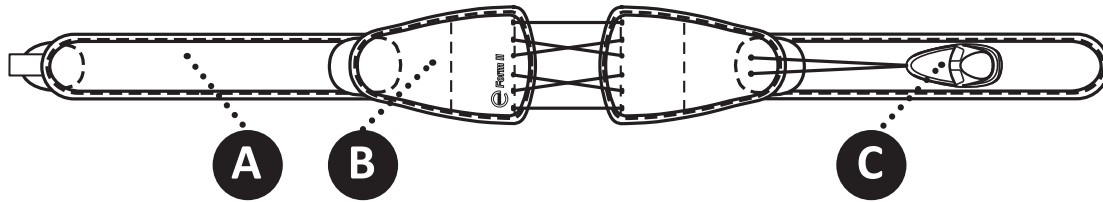

Application of Brace:

1 SIZE BELT. Take circumferential measurement. Select larger brace for in between sizes.



TRIM EXCESS material on Belt Wings (A) if needed.

2 LOOSEN BRACE by detaching the Pull Tab (C) and pull Rear Panel Section (B) in opposite directions.



3 APPLY BRACE by centering Rear Panel Section just above buttock crease. Wrap Belt Wings in front about 1" (2.5 cm) above widest part of hips and secure hook and loop.



4 TIGHTEN BRACE by grasping Pull Tab and pulling away from body. Secure Pull Tab to Belt Wing.



Brace Adjustment:

OPTIMAL WAIST FIT: Detachable Belt Wings can be repositioned for improved patient fit.

BELT PADS: Slide Belt Wing through elastic strap on Belt Pad so anti-slip material is facing patient. Repeat on the opposite Belt Wing.



For more information including application videos, please visit

DJOglobal.com/products/exos/exos-form-ii-back-brace or call **888.405.3251**

MATERIAL CONTENTS: Nylon, Polyethylene, Polyester, Antimicrobial, Polyurethane, Polyvinylchloride, Silicone, Elastic. **FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.**

CONTRAINDICATIONS: Pulmonary, cardiovascular or skeletal conditions which have risk to be made worse as a result of compression and/or pressure; unstable, displaced fractures.

For product assistance, please contact Product Support at **1.888.405.3251** or email product.specialist@djoglobal.com

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

DJO, LLC | A DJO Global Company
 T 800.336.6569 D 760.727.1280 F 800.936.6569
 1430 Decision Street | Vista, CA 92081-8553 | U.S.A.
DJOglobal.com/exos

U.S.A. and International Patents Pending ©2016
 DJO, LLC 00-4184 Rev A



MOTION IS MEDICINE™