ENGLISH

MD Medical Device

INTENDED USE
The device is intended to support drop-foot.

Indications for use
Drop foot.
No known contraindications.

Warnings and Cautions:
If you have a medical condition that decreases blood circulation in your limbs, then consult your healthcare professional before you use the device. Such medical conditions include, but are not limited to, diabetes or peripheral vascular disease.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference. Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities. The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.

If any pain, skin irritation, or unusual reaction occurs with the use of the device. The device is for single patient – multiple use.

FITTING INSTRUCTIONS

Device Application
There are two versions, with the inlay (A) for in-shoe use and the foot wrap (B).

- Measure the circumference of the ankle around the malleoli (Fig. 1a).
- Detach the inlay from the ankle strap by pressing the button in the middle of the quick-acting closure.

- If using the optional foot wrap, also measure the foot immediately behind the ball of the foot (Fig. 1b).
- Apply the ankle strap with the narrow hook and loop strip at the bottom and the quick-acting closure at the front. Place the ankle strap above the widest part of the ankle. Fasten the ankle strap tightly but not constrict the ankle (Fig. 2a, 2b).

- Placing the attachments:
- Place the inlay between the tongue of the shoelaces. Then pull the elastic strap between the two lace lines (Fig. 3a). By pulling the inlay further in the direction of the front of the foot, it provides more support when raising the foot. Make sure the elastic strap is also pulled between the lace lines further down the foot.

- Place the foot wrap immediately behind the ball of the foot so that the button rests on top (Fig. 3b).
- Connect the ankle strap to the inlay (A) or foot wrap (B) by pushing the two parts of the quick-acting closure together (Fig. 4).

Accessories and Replacement Parts
Please refer to the Ossur catalog for a list of available replacement parts or accessories.

USAGE

Cleaning and care
• Hand-wash using mild detergent and rinse thoroughly.
• Air dry.

Note: Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener. **Note:** Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

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DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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