

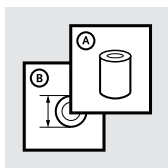
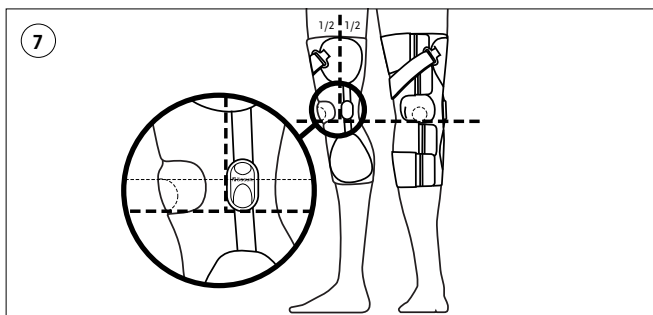
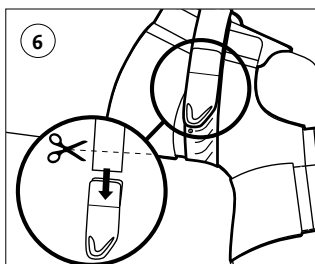
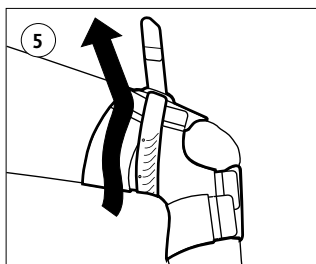
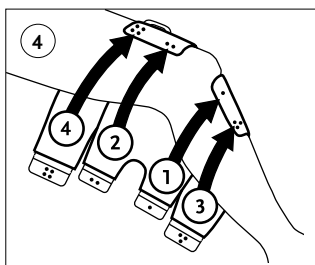
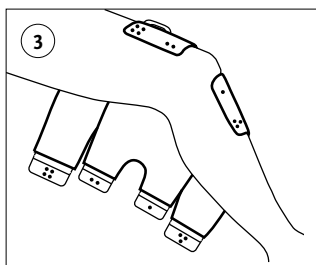
Instructions for Use

FORMFIT[®] OA EASE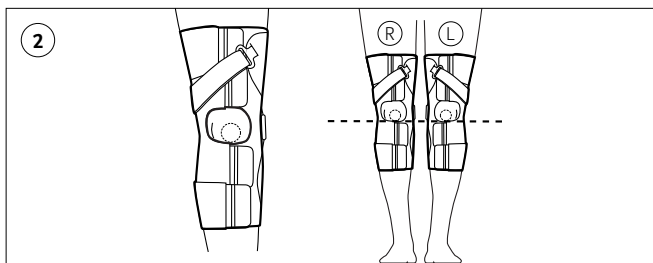

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	7
FR	Notice d'utilisation	11
ES	Instrucciones para el uso	15
SV	Bruksanvisning	19
NL	Gebruiksaanwijzing	22
PL	Instrukcja użytkowania	25
ZH	中文说明书	29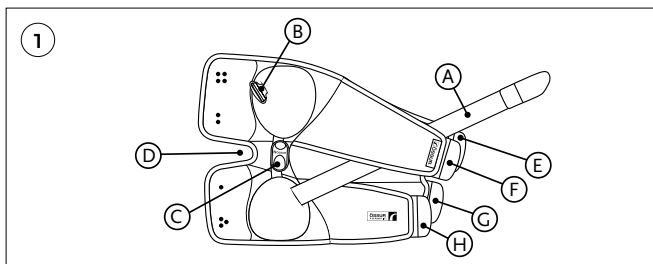

ENGLISH

PRODUCT DESCRIPTION

Your FormFit® OA Ease brace has a sleeve with a frame and a dynamic force system (DFS) strap. The sleeve, frame and DFS strap function work together like three levers. They unload your knee when you use it. This removes pressure from the side of the knee that hurts.

Sizes

The FormFit® OA Ease is available in four versions with seven different sizes.

Accessories

Accessories are available separately. You can use the extension kits and calf and thigh liners to make your brace fit better.

If necessary, a certified prosthetist orthotist will use the flexion/extension stop kit to adjust range of motion.

Accessory	Catalogue number
Suspension kit	OAW00007
OA Ease calf liner, standard	FOA000101
OA Ease calf liner, extra	FOA000102
OA Ease thigh liner, standard	FOA000201
OA Ease thigh liner, extra	FOA000202
OA Ease SM extension	FOA000301
OA Ease SM extension, 10 pieces	FOA000302
OA Ease LG extension, XS – SM RM/LL	FOA000401
OA Ease LG Extension, MD – LG RM/LL	FOA000402
OA Ease LG Extension, XL – XXXL RM/LL	FOA000403
OA Ease LG extension, XS – SM LM/RL	FOA000404
OA Ease LG Extension, MD – LG LM/RL	FOA000405
OA Ease LG Extension, XL – XXXL LM/RL	FOA000406
Flexion/ Extension Stop Kit	B-301040047
Wash bag kit	WB000001

PRODUCT OVERVIEW

The FormFit® OA Ease has these parts (**fig. 1**):

- A. Dynamic force system (DFS) strap
- B. D-Ring
- C. Hinge
- D. Kneecap opening
- E.-H. Tabs

INDICATIONS FOR USE

- Uni-compartmental osteoarthritis of the knee (KOA)
- Other conditions that may benefit from uni-compartmental unloading, such as degenerative meniscal tears

CONTRAINDICATIONS

None known.

INTENDED USE

The FormFit® OA Ease is non-invasive, single patient, reusable knee brace. It is used in direct contact with the skin.

The support is intended to be used during activities of daily living where user can benefit from uni-compartmental unloading of the knee.

GENERAL SAFETY INSTRUCTIONS

CAUTION:

- For patients with peripheral vascular disease, neuropathy and sensitive skin, physician supervision is recommended.
- Only use the device for a single patient. Do not use for multiple patients.

USING THE BRACE

NOTICE: If redness appears, decrease use of the brace. Slowly increase use to let the skin adapt.

Positioning the Brace

1. Sit down on a chair.
2. Put your foot flat on the floor and bend your knee to a 30° angle.
3. Fully open the brace.
4. Position the hinge on the side that hurts.

NOTE: The hinge must be on the correct side. If the hinge is not, the brace will not work.

5. Put the kneecap opening over your kneecap (**fig. 3**).
6. Make sure the lower edge of the kneecap opening is below your kneecap (**fig 2**).
7. Also make sure the front side of the hinge is on the midline of your leg.

Attaching the Tabs

1. Keep the brace in place with one hand.
2. Use your other hand to attach the tabs in the correct locations and sequence (**fig. 4**).

NOTE: The dots show you the locations where to attach the tabs. The dots also give you the correct sequence for attaching the tabs.

3. Make sure all the tabs are tight, but comfortable. If necessary, adjust the tabs again.

Using the Dynamic Force System Strap

1. Put your foot flat on the floor and bend your knee to an 80° angle.
2. Move the dynamic force system (DFS) strap behind your leg and put it through the D-ring (**fig. 5**).
3. Keep the brace in place with one hand.
4. Use the other hand to tighten the DFS strap.

NOTE: You must tighten the DFS strap until pain relief is sufficient.

Shortening the Dynamic Force System Strap

1. Remove the alligator tab from the end of the dynamic force system (DFS) strap.
2. Fasten the alligator tab at “5” on the DFS strap (**fig. 6**).
3. Pull the loose end of the DFS strap down over the alligator tab.
4. When sufficiently tight, attach the DFS strap to the alligator tab. You will see the strap is too long and can be shortened (**fig. 6**).
NOTICE: If the strap is too short, the brace can be too tight. If the brace is too tight, you will not be able to use it.
5. Use scissors to shorten the strap as applicable.

Examining the Brace Position

Examine the position of the brace after you put it on (**fig. 7**). Make sure it sits tightly on the knee and gives sufficient pain relief.

PRODUCT MATERIAL LIST

Nylon, polyurethane, thermoplastic elastomer, acrylonitrile butadiene styrene, ethylene-vinyl acetate, polyester, steel, fiber glass reinforced nylon, polypropylene, aluminum and elastane.

CLEANING YOUR BRACE

NOTICE: Fasten the strap ends before you wash the brace. If you do not fasten the strap ends, they can damage the brace materials.

Fully put the brace into a washbag. Do not remove the sleeve or the strap.

Machine wash on a gentle cycle at max 30°C with a mild detergent. Leave to air dry. Do not tumble dry.

NOTICE:

- Avoid contact with chlorinated water or saltwater. In case of contact, rinse well with tap water.
- Do not wash using bleach. Bleach will decrease product life and can cause skin irritation.
- Do not wash using a fabric softener.

IMPORTANT NOTICE TO USERS AND/OR PATIENTS ESTABLISHED IN EUROPE:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations. Users should contact their local governmental office for information on how these items can be disposed of in an environmentally sound manner.

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Zubehör	Katalognummer
Aufhängungsgurt-Set	OAW00007
OA Ease Unterschenkelpolster, Standard	FOA000101
OA Ease Unterschenkelpolster, extra	FOA000102
OA Ease Oberschenkelpolster, Standard	FOA000201
OA Ease Oberschenkelpolster, extra	FOA000202
OA Ease SM-Extension	FOA000301
OA Ease SM-Extension, 10 Stück	FOA000302
OA Ease LG-Extension, XS – SM RM/LL	FOA000401
OA Ease LG-Extension, MD – LG RM/LL	FOA000402
OA Ease LG-Extension, XL – XXXL RM/LL	FOA000403
OA Ease LG-Extension, XS – SM LM/RL	FOA000404
OA Ease LG-Extension, MD – LG LM/RL	FOA000405
OA Ease LG-Extension, XL – XXXL LM/RL	FOA000406
Flexions-/Extensionsbegrenzung-Set	B-301040047
Wäschebeutel-Set	WB000001

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Accessoire	Référence
Kit de suspension	OAW00007
Capitonnage de mollet OA Ease, standard	FOA000101
Capitonnage de mollet OA Ease, extra	FOA000102
Capitonnage de cuisse OA Ease, standard	FOA000201
Capitonnage de cuisse OA Ease, extra	FOA000202
Extension OA Ease Small	FOA000301
Extension OA Ease Small, 10 pièces	FOA000302
Extension OA Ease Large, XS – S MD/LG	FOA000401
Extension OA Ease Large, M – L MD/LG	FOA000402
Extension OA Ease Large, XL – XXXL MD/LG	FOA000403
Extension OA Ease Large, XS – SM MG/LD	FOA000404
Extension OA Ease Large, MD – LG MG/LD	FOA000405
Extension OA Ease Large, XL – XXXL MG/LD	FOA000406
Kit de butée de flexion/d'extension	B-301040047
Kit sac de lavage	WB000001

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