

Push[®] med Knee Brace

Optimal knee stabilization, less bulk

- Follows natural rotational knee axis using unique non-axial leaf spring hinges which remain close to the joint
- Provides substantial medio-lateral knee support
- Prevents brace slippage via internal silicone strips
- Streamlined contour allows brace to be worn under clothing
- Can be machine washed on gentle cycle at low temperature
- Creates soft, comfortable feel and wicks moisture through the Sympress[™] microfibers



Indications

- Osteoarthritis/rheumatoid arthritis (with or without swelling)
- Instability after trauma/injury to collateral ligaments(s)
- Post-operative protection after collateral ligament surgery

Measuring / Sizing

Bend your knee 90° and measure around the narrowest part of your knee.



SIZE	INCHES	CENTIMETERS
1	11 - 12 1/4	28 - 31
2	12 1/4 - 13 1/8	31 - 34
3	13 1/8 - 14 1/2	34 - 37
4	14 1/2 - 16 1/8	37 - 41
5	16 1/8 - 17 3/8	41 - 45




Easy application!

Hold both hinges on both sides to pull the brace up until the circular stitching is positioned over your kneecap



Close strap (1) around back of thigh and strap (2) around front of thigh. Adjust to assure hinges are in the middle on each side

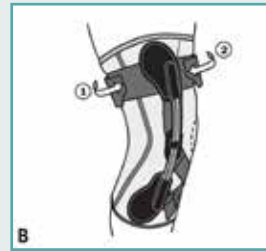

Close the overlapping diagonal straps (3) and (4)



Insurance Information

- An L-code application for the Push med Knee Brace is currently in progress.
- BraceLab is not a provider. We do not file insurance nor can we provide information on insurance coverage or reimbursement rates. Please contact your insurance for filing information.