QUICK START GUIDE & WARNINGS

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**WIRELESS PROFESSIONAL 4CH STD/FULL 2CH
REHAB/THETA/PHYSIO**

QUICK START GUIDE & WARNINGS - CONTENT

QUICK START GUIDE

WIRELESS PROFESSIONAL 4CH STD/FULL 2CH.....	4
REHAB/THETA/PHYSIO	9

RULES FOR USE

RULES FOR USE - EN	14
REGLES D'UTILISATION - FR.....	16
REGELN ZUR NUTZUNG - DE.....	18
NORMAS DE USO - ES.....	20
REGRAS DE UTILIZACAO - PT.....	22
REGELS VOOR HET GEBRUIK - NL.....	24
REGOLE PER L'USO - IT.....	26
KÄYTTÖOHJEET - FI.....	28
REGLER FOR BRUK - NO.....	29
REGLER FÖR ANVÄNDANDE - SV.....	30
REGLER FOR BRUG - DA.....	31
PRAVIDLA UŽÍVÁNÍ - CZ.....	32
KULLANIM KURALLAR - TK.....	33
ΚΑΝΟΝΕΣ ΛΕΙΤΟΥΡΓΙΑΣ - EL.....	34
ПРАВИЛА ИСПОЛЬЗОВАНИЯ - RU.....	36
ZASADY UŻYTKOWANIA - PL.....	38

IMPORTANT INFORMATION


IMPORTANT INFORMATION - EN	40
INFORMATIONS IMPORTANTES - FR.....	46
WICHTIGE INFORMATIONEN - DE.....	52
INFORMACION IMPORTANTE - ES.....	58
INFORMACION IMPORTANTES - PT.....	64
BELANGRIJKE INFORMATIE - NL.....	70
INFORMAZIONI IMPORTANTI - IT.....	76
TÄRKEÄT OHJEET - FI.....	82
VIKTIG INFORMASJON - NO.....	88
VIKTIG INFORMATION - SV.....	93
VIGTIG INFORMATION - DA.....	98
DŮLEŽITÁ INFORMACE - CZ.....	103
ÖNEMLI BİLGİLER - TK.....	109
ΣΗΜΑΝΤΙΚΕΣ ΠΛΗΡΟΦΟΡΙΕΣ - EL.....	115
ВАЖНАЯ ИНФОРМАЦИЯ - RU.....	122
WAŻNE INFORMACJE - PL.....	129

WIRELESS PROFESSIONAL 4CH STD/FULL | 2CH

1 CHARGING

Charger – Aufladen – Cargando – Carregamento – Opladen – Caricamento – Lataaminen – Lade – Uppladning – Opladning – Nabíjeň – Šarj – φορτίση Зарядка батареи Ładowanie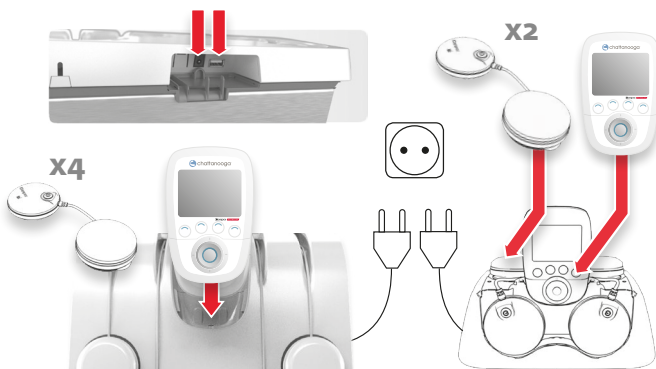

2 SYNCHRONISING (4 CH DOCKING STATION ONLY)


Synchroniser – Synchronisation – Sincronizando Sincronizar – Synchronisatie – Sincronizzazione – Synkronointi – Synkronisering – Synkronisera – Synkronisering – Synchronizace – Senkronizasyon – συγχρονισμος – Синхронизация Synchronizacja

(Station d'accueil smart 4CH uniquement – Smart 4CH Ladestation - Únicamente estación de carga 4 canales - Estação smart 4ch unico - Smart 4CH Docking Station - Solo Smart Docking Station 4ch - Vain 4-kanavaisen matausyksikkö - Smart 4 kanal ladestasjon - använd enbart smart 4 kanals laddaren - Smart 4 kanals ladestation - Pouze dokovací stanice SMART 4 kanály - Sadece 4 kanallı akıllı istasyon - μόνο εξυπνη 4καναλη βαση συνδεση - 4-канальная док-станция - Tylko stacja dokująca SMART 4CH)

3 POWER ON

Allumer – Einschalten – Conectar – Ligar
 – Inschakelen – Accensione Virta päällä
 Strøm på Slå på Power on – Zapnout
 Cihazı Aç σε λειτουργία – Включение
 Włączyć zasilanie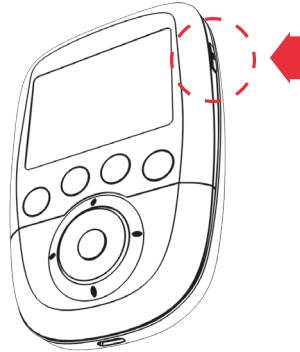

4 SELECT LANGUAGE (FIRST USE ONLY)

Choisir la langue (Première utilisation
 uniquement) – Sprache auswählen (Erster
 Gebrauch) - Seleccionar idioma (Primer uso
 únicamente) - Escolher o idioma (Primeira
 utilização unica) - Selecteer de taal (Enkel
 bij eerste gebruik) - Selezionare la lingua
 (Solo Al primo utilizzo) - Valitse kieli (Vain
 ensimmäisellä käytöllä) - Velg språk (Første
 bruk Vid) - Välj språk (Första användningen)-
 Vælg sprog (Første brug) - Vyberte jazyk
 (Pouze první použití) - Dili Seçin (Sadece ilk
 kullanım) - επιλέξτε γλώσσα (μονο για την
 πρώτη χρήση) - Выберите язык (Первое
 использование) - Wybrać język (Tylko do
 jednorazowego użytku)

START A PROGRAMME

Démarrer un programme – Ein Programm starten - Iniciar un programa - Começar um programa - Start het programma - Avvio di un programma - Käynnistä ohjelma - Start program - Starta programmet - Start et program - Spustit program - Bi program başlatin - εκκίνηση ενος προγραμματος - Запуск программы стимуляции - Włączyć program

5 CHOOSING A PROGRAMME CATEGORY

Choisir une catégorie de programme
 – Eine Programmkategorie auswählen
 - Seleccionar una categoría de programa - Een programmacategorie kiezen - Seleziona Categoria - Valitse kategoria - Velg program kategori - Välj programkategori - Vælg et program kategori - Vybrat kategorii program - Bir program kategorisi seçin - επιλέξτε κατηγορία προγραμματος - Выбрать категорию программы - Wybrać kategorię programu


6 CHOOSING A PROGRAMME

Choisir un programme - Ein Programm auswählen - Seleccionar un programa - Escolher um programa - Kies een programma - Seleziona Programma - Valitse ohjelma - velg et program - Välj program - Vælg et program - Vybrat program - Program seç - επιλέξτε προγραμμα - Выбрать программу - Wybrać program


7 ELECTRODE AND MODULE SETUP

Mise en place électrodes et modules – Einstellung Elektroden und Module - Electrodos e instalación módulos 6 Posicionamento dos eléctrodos e do módulo - Electrode en module setup - Setup elettrodo e modulo - Elektroodin ja modulini asetukset - Elektrode og moduloppsett - Elektrod och modul placering - Elektrode og modul setup - Nastavení elektrod a modulu - Elektrot ve modül ayarlaması - ρυθμίση ηλεκτροδίων και μονάδας - Установка электродов и модуля - Wyregulować elektrody i moduły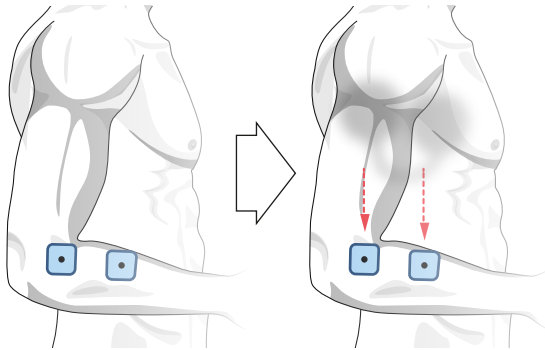

8 TURN ON MODULE (S)

Allumer le(s) module(s) – Modul einschalten - Conectar los módulos - Ligar o modulo - De modules inschakelen - Attivare il modulo - Käynnistä moduli - Slå på modulen - Slå på modulerna - Tænd modulen - Zapnout modul - Modülü aç - ενεργοποίηση μονάδας - Включение модуля - Włączyć moduł

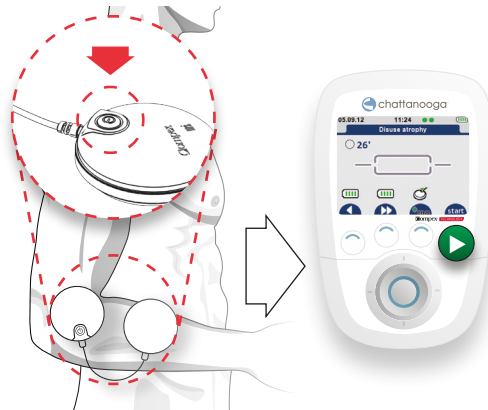9 INCREASE THE INTENSITY TO START STIMULATION

Augmenter l'intensité pour démarrer la stimulation
 - Steigerung der Stimulationsintensität um die Stimulation zu starten - Aumentar la intensidad para iniciar la estimulación - Aumentar a intensidade para començar a estimulação - Verhoog de intensiteit om de stimulatie te starten - aumentare intensità per far partire la stimolazione - Kasvata intensiteettiä käynnistääksesi stimulaatio - Velg intensitet og start stimulering - öka intensiteten för att starta stimuleringen - Øg intensiteten for at starte stimulationen - Zvýšit intenzitu pro začátek stimulaace - Stimúlasyona bařlamak için akımı arttır - αυξηστε την ενταση για έναρξη διεγερση - Увеличить интенсивность для начала стимуляции - Zwiększyć natężenie i rozpocząć stymulację

10 END OF PROGRAMME

Fin du programme – Program beenden - Fin del programa - Fim do programa- Einde van het programma - Fine di un programma - Ohjelman loppu -Slutt på program - Avsluta programmet - Program slut - Ukončit program - Program Sonu - τερματισμος προγραμματος - Завершение программы стимуляции -Zakończyć program

REHAB/THETA/PHYSIO

1 CHARGING

Charger – Aufladen – Cargando – Carregamento – Opladen – Caricamento – Lataaminen – Lade – Uppladdning – Opladning – Nabíjení – Şarj – φορτίση
Зарядка батареи – ładowanie

2 POWER ON

Allumer – Einschalten – Conectar – Ligar – Inschakelen – Accensione – Virta päällä – Strøm på – Slå på – Power on – Zapnout – Cihazı Aç – σε λειτουργία – Включение – Włączyć zasilanie

3 SELECT LANGUAGE (First use only)

Choisir la langue (Première utilisation uniquement) – Sprache auswählen (Erster Gebrauch) – Seleccionar idioma (Primer uso únicamente) – Escolher o idioma (Primeira utilização única) – Selecteer de taal (Enkel bij eerste gebruik) – Selezionare la lingua (Solo Al primo utilizzo) – Valitse kieli (Vain ensimmäisellä käytöllä) – Velg språk (Første bruk Vid) – Välj språk (Första användningen) – Vælg sprog (Første brug) – Vybírete jazyk (Pouze první použití) – Dili Seçin (Sadece ilk kullanım) – επιλέξτε γλώσσα (μονο για την πρώτη χρήση) – Выберите язык (Первое использование) – Wybrać język (Tylko do jednorazowego użyciu)

START A PROGRAMME

Démarrer un programme - Ein Programm starten - Iniciar un programa - Começar um programa - Start het programma - Avvio di un programma - käynnistä ohjelma - Start program - Starta programmet - Start et program - Spusti program - Bi program başlatın - εκκίνηση ενος προγραμματος - Запуск программы стимуляции - Włączyć program

4 CONNECT CABLES AND ELECTRODES

Connecter les câbles et électrodes - Kabel und Elektroden anschließen - Conectar cables y electrodos - Ligar os cabos e electrodos - Sluit de kabels en elektroden aan - Collegare cavi ed elettrodi - Kiinnittää kaapelit ja elektrodit - Koble til kabler og elektroder - Anslut kablar och elektroder - Tilslut kabler og elektroder - Připojit kabely a elektrody - Kablo ve elektrotları bağla - συνδέστε καλώδια και ηλεκτρόδια - Подключите кабели и электроды - Podłączyć kable i elektrody

5 CHOOSING A TYPE OF TREATMENT

Choisir un type de traitement - Eine Therapiemethode auswählen
 Seleccionar un tipo de tratamiento - Escolher um tipo de tratamentos - Kies een type van behandeling - Scegliere un tipo di trattamento - Hoitotyyppin valinta - Velg type behandling - Välj typ av behandling - Vælg en type af behandling - Vyběr terapie - Bir tedavi şekli seçimi - επιλεγοντας τυπο θεραπειας - Выбор типа лечения - Wybrać rodzaj leczenia

6 CHOOSING A PROGRAMME CATEGORY

Choisir une catégorie de programme - Eine Programmkategorie auswählen - Seleccionar una categoría de programa - Escolher uma categoria de programa - Een programmacategorie kiezen - Seleziona Categoria - Valitse kategoria - Velg program kategori - Välj programkategori - Vælg et program kategori - Vybrat kategorii program - Bir program kategorisi için - επιλέξτε κατηγορία προγραμματος - Выбрать категорию программы - Wybrać kategorię programu

7 CHOOSING A PROGRAMME

Choisir un programme - Ein Programm auswählen
 - Seleccionar un programa - Escolher um programa
 - Kies een programma - Seleziona Programma -
 Valitse ohjelma - velg et program - Välj program
 - Vælg et program - Vybrat program - Program
 seç - επιλέξτε προγράμμα - Выбрать программу -
 Wybrać program

8 CUSTOMISING A PROGRAMME

Personnaliser le programme - Ein Programm
 personalisieren - Personalizar un programa -
 Personalizarb um programa - Personaliseer een
 programma - Personalizzare un programma
 - Ohjelman customointi - Tilpass et program -
 Programmera - Lav eget program - Upravit program
 - Bir programın kişiselleştirilmesi - προσαρμόζοντας
 ένα προγράμμα - Настройка программы -
 Dostosować program do indywidualnych potrzeb

9 INCREASE THE INTENSITY TO START STIMULATION

Augmenter l'intensité pour démarrer la stimulation
 - Steigerung der Stimulationsintensität um die Stimulation zu starten - Aumentar la intensidad para iniciar la estimulación - Aumentar a intensidade para començar a estimulação - Verhoog de intensiteit om de stimulatie te starten - Aumentare intensità per far partire la stimolazione - Kasvata intensiteettiä käynnistääkseen stimulaatio - Velg intensitet og start stimulering - öka intensiteten för att starta stimuleringen - Øg intensiteten for at starte stimulationen - Ζυγίσιτ ένζεντιζυ προ ζαζάτεκ stimulaçe - Stimúlasyona başlamak için akımı arttır - αυξήστε την ένταση για ενάρξη διεγέρση - Увеличить интенсивность для начала стимуляции - Zwiększyć natężenie i rozpocząć stymulację

10 END OF PROGRAMME

Fin du programme - Programm beenden
 - Fin del programa - Fim do programa
 - Einde van het programma - Fine di un programma - Ohjelman loppu - Slutt på program - Avsluta programmet - Program slut - Ukončit program - Program Sonu - τερματισμός προγράμματος - Завершение программы стимуляции - Zakończyc program

Do not get the stimulator in contact with massage oils or any other products of the same type that could damaged the device or its accessories.

Placement of electrodes

The placements of the electrodes depend on the indications being treated. Depending on the characteristics of the current used for each programme, the electrode connected to the positive pole may benefit from a "prime" location that is likely to increase the efficacy of the treatment. This is the case particularly for muscular electrostimulation programmes requiring strong muscular contractions, for which it is recommended that the electrode with positive polarity is placed on the motor point of the muscle. These recommended positions are also the optimal positions for the mi sensor system, and as such should be followed closely. The choice of electrode size (large or small) and the correct positioning of the electrodes on the muscle group that needs to be stimulated are determining factors and are essential for stimulation to be effective. As a result, always use the size of the electrodes shown on the images. Unless advised otherwise by a doctor, always follow the positions specified on the images.

Body position

To determine the stimulation position to be used based on the position of the electrodes and the programme chosen, please refer to the images of where the electrodes are positioned. The position of the person to be stimulated depends on the muscle group that requires stimulation and on the programme chosen. For programmes requiring muscle contractions (tetanic contractions), working the muscle isometrically is always recommended to prevent cramps and muscle soreness after the session. For example, when the quadriceps are stimulated, the patient will be placed in a seated position with the ankles fixed with straps to prevent the knees extending. For other types of programmes (for example, analgesic programmes), which do not cause muscle contractions, position the patient as comfortably as possible.

Adjusting stimulation intensity

Stimulation energy settings (intensity level)

For programmes which cause muscle contractions, it is important to use the maximum stimulation intensity, i.e. always at the limit of what the patient is able to tolerate. This means that, in a stimulated muscle, the number of fibres working depends on the stimulation intensity. The maximum stimulation intensity must therefore be used in order to engage as many fibres as possible. Below this significant stimulation intensity, the number of fibres engaged in the stimulated muscle is too low to considerably improve the quality of the muscles. The maximum intensity will not be reached during the first session but after at least 3 sessions, during which the ability to produce strong muscle contractions will be increased gradually so that the patient becomes accustomed to electrostimulation. After the warmup, which should produce clear muscle twitching, the stimulation intensity must be increased progressively contraction by contraction throughout the work sequence.

If your device emits a beeping sound and the + symbols under the active channels begin to flash, the stimulator is suggesting you increase the level of the stimulation energies. If you are working at the patient's maximum tolerance level, simply ignore this message. The intensity used should also be increased session by session.

RULES FOR USE - EN

For TENS treatments, stimulation is only sensory. The intensity must therefore be increased until the patient has a pins and needles sensation (tingling) that is not considered painful.

For neuromuscular electrostimulation programmes which do not cause tetanic muscle contractions (frequencies $< 10\text{Hz}$), the energies must be increased gradually until muscle twitching is produced that can be clearly seen or felt.

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Fields of application

The Rehab/Theta/Physio/Wireless Professional is a stimulator designed for use by health professionals to ensure electric stimulation treatments in pain management (TENS) neuro muscular stimulation (EMS/ NMES). The Physio device also allows Direct applications (Iontophoresis/Hyperhidrosis/Oedema) and Denervated muscles stimulation.

The Rehab/Theta/Physio/Wireless Professional unit is an important supplement to medical and therapeutic treatment for use in hospitals, clinics, general practices and at patient's home by a therapist.

When prescribed by a health care professional, Rehab or Theta devices can be used by patients at home without the presence of the health care professional. Patients shall read carefully this manual and contact their health care professional or DJO in case of any question.

Contraindications

ContraIndications: Do NOT use the Rehab/Theta/Physio/Wireless Professional on patients with:

- Implanted electronic devices. Do not use the device if you have a cardiac stimulator, implanted defibrillator or other implanted electronic/electrical device.
- Epilepsy
- Pregnancy (do not use on abdominal region)
- Serious arterial circulation problems in lower limbs
- Abdominal or inguinal hernia
- Do not use chest stimulation on patients with cardiac arrhythmia

This could cause an electrical shock, burns, electrical interference or death.

Safety information

Danger

Explosion hazard — Rehab/Theta/Physio/Wireless Professional is not designed for use in areas where an explosion hazard may occur. An explosion hazard may result from the use of flammable anesthetics, oxygenrich environments, skin cleansing agents and disinfectants.

Warning

Patient hazard:

- Only authorised individuals are allowed to operate the Rehab/Theta/Physio/Wireless Professional. Individuals are authorised after receiving training in the operation of the unit and reading this operating on manual.
- Before using the therapy unit, the operator must ascertain that it is in correct working order and operating condition. The cables and connectors, in particular, must be checked for signs of damage. Damaged parts must be replaced immediately, before use.
- Stop therapy immediately if you have doubts about the device settings and/or the therapy protocol.

IMPORTANT INFORMATION - EN

- Patients must be **fully conscious** while being instructed in the use of the therapy unit and during therapy.
- The choice of the **therapy parameters** to program and of the **therapy protocols** to use is **restricted to the responsible physician or therapist**. It is the physician's or therapist's decision whether or not to use the unit on a specific patient.
- The patient must be familiar with the functions of the Rehab/Theta/Physio/ Wireless Professionaldevice allowing them to stop therapy, if needed. **Patients unable to operate the emergency stop function** e.g. paralytic patients, must never be left unattended during therapy.
- **Any accessories used with Rehab/Theta/Physio/Wireless Professional** must first be approved by the manufacturer. Using of not specified accessories and detachable parts could be unsafe
- The **utmost caution is advised** under the following conditions. Depending on the judgement of the responsible physician, the unit may only be applied under supervision and with the parameters defined by the responsible physician. Otherwise the exercise may be too strenuous for the patients with:
 1. hypertension (> stage 2), ischemic heart disease and cerebrovascular diseases
 2. cardiovascular diseases
 3. pregnancy
 4. under 16 years of age
- Never apply the electrodes:
 - Near the head or on the eyes
 - On the front and side of the neck (especially carotid sinus)
 - Counterlaterally, i.e. do not use two poles connected to the same channel on opposite sides of the body.
 - On or near skin lesions of any kind (wounds, swelling, burns, irritation, eczema, cancerous lesion, etc.)
 - Crossing over the heart
- If the person is pregnant or menstruating do not place electrodes directly on the uterus area or connect pairs of electrodes on either side of the abdomen to avoid any risk for the mother and/or the baby.
- Never allow muscular contraction during a stimulation session to result in movement. You should always stimulate isometrically; this means that the extremities of the limb in which a muscle is being stimulated must be firmly fixed, so as to prevent any movement that results from contraction.
- Extreme caution should be taken when in use around small children and babies! Sufficient distance to the device and its accessories is mandatory for their safety!
- Never leave the device unattended when it is switched on!
- After use, store the device in a safe place to avoid other people not informed to use the device;
- This device is not a toy but a medical device. Misunderstanding its use can cause damage!

IMPORTANT INFORMATION - EN

Shock hazard – Strictly observe the following warnings. Failure to do so could endanger the lives of the patient, the user and other persons involved.

- **Before use** allow the Rehab/Theta/Physio/Wireless Professionalto reach room temperature. If the unit has been transported at temperatures below 0 °C (32°F), leave it to reach at room temperature for about 2 hours, until any condensation has disappeared.
- **Electrosurgical equipment or defibrillators.** Disconnect the electrodes from the device before using electrosurgical equipment, or a defibrillator, to avoid cutaneous burns from the electrodes and destroying the device.
- **Electronic surveillance equipment.** Do not apply stimulation near electronic surveillance equipment (e.g. cardiac monitors, ECG alarms), as there is a risk they may not work properly whilst the electrical stimulation device is being used.
- **Electromagnetic radiation.** Do not use the stimulator in areas in which unprotected devices are used to emit electromagnetic radiation. Portablecommunications equipment can interfere with the device.
- **Cancer.** Do not apply stimulation if you have progressive cancer or near any cancerous tumour. The increased metabolism, caused by certain modes of stimulation, is likely to encourage cancer cells to spread
- **Muscle shortening.** During the muscular contraction phase it is recommended to hold the extremities of the stimulated limbs to avoid any shortening of the muscle during contraction, which could cause cramps.
- **Contralateral stimulation.** Do not use two terminals connected to the same channel on opposite segments of the body (for example, a positive terminal on the left arm and a negative terminal on the right arm).
- **Loss of sensation.** Proceed with caution if stimulation is applied to areas of the skin whose level of sensation is lower than normal. Do not apply stimulation to a person who cannot express themselves.
- **Battery leakage.** If there is leak from a component, take steps to ensure the liquid does not come into contact with skin or eyes. Should this occur, wash the affected area with water and consult a doctor.
- **Strangulation.** Do not wind cables around the neck. Tangled cables can cause strangulation.
- **Postsurgery.** Proceed with caution after recent surgery.
- **Accessibility of the power adaptor.** The plug socket must be close to the power adaptor and be easily accessible.
- **Internal bleeding.** Proceed with caution if you are prone to internal bleeding; for example, after an injury or a fracture.
- The Rehab/Theta/Physio/Wireless Professionalmust only be operated in dry rooms.
- **Do not use** the Rehab/Theta/Physio/Wireless Professionalin **water or in a humid atmosphere** (sauna, bath, shower etc.)
- When connecting the unit to other equipment or when creating a medical system, check that the sum of leakage currents will not cause any hazard. Please contact DJO GLOBAL if you have questionsregarding this matter.
- No modification of this equipment is allowed.
- Do not open the product and its accessories as there is risk of electrocution.

IMPORTANT INFORMATION - EN

- Before cleaning and service interventions, **turn the device off.**
- **Liquids and foreign material (such as dust, metal etc.) must not be allowed** to enter the device. If such material has entered into the units, it must be immediately checked by a service technician, before it can be reused.
- **Electricity supply.** Never connect the stimulation cables to an external power supply, as there is a risk of electrocution
- Do not apply stimulation near the area of an implant, such as cochlear implants, pacemakers, skeletal anchorage or electric implants. This could cause an electrical shock, burns, electrical interference or death
- Never use the Rehab/Theta/Physio/Wireless Professional or the AC adaptor if it is damaged or open. There is a risk of electric shock.
- Disconnect the AC adaptor immediately if there is abnormal heating or smell, or if smoke comes from the AC adaptor or the device.
- Equipment malfunction – this warnings can cause equipment malfunctions that result in patient hazards
- Magnetic and electrical fields are capable of interfering with the proper performance of the unit. For this reason make sure that all external devices operated in the vicinity of the unit comply with the relevant EMC requirements. → Xray equipment, MRI devices, →radio systems and cell phones are possible sources of interference as they may emit higher levels of electromagnetic → radiation.
- Keep the unit away from such equipment and verify its performance before use.
- Do not use the Rehab/Theta/Physio/Wireless Professional within one meter of short wave or microwave devices as this could alter the currents generated by the stimulator. If you are in any doubt as to the use of the stimulator in close proximity to another medical device, seek advice from the manufacturer of the latter or from your doctor.
- Exercise caution when using electrotherapy while the patient is connected to monitoring equipment with electrodes attached to the body. Stimulation could disrupt the signals sent to the monitoring equipment.
- Refer repair and maintenance to authorized persons. Persons are authorized after training by a specialist trained and commissioned by the manufacturer.
- Inspect the Rehab/Theta/Physio/Wireless Professional and its accessories for damage and loose connections at least once a year. Damaged and worn parts must be immediately replaced with original spare parts by authorized staff.
- Do not use the device if you are connected to a high frequency surgical instrument as this could cause skin irritation or burns under the electrodes

IMPORTANT INFORMATION - EN

Caution

- Patient hazard – these cautions need to be observed to avoid the risk of electrical shock or other negative effects to the patient.
- Do not apply stimulation close to metal. Remove jewellery, piercings, belt buckles or any other metallic product or device in the area of stimulation.
- Be careful if the patient has sensitivity problems or is not able to communicate that he or she feels discomfort, however light.
- Never begin an initial stimulation session on a person who is standing. The first five minutes of stimulation must always be performed on a person who is sitting or lying down. In rare instances, people of a nervous disposition may experience a vasovagal reaction. This is of psychological origin and is connected with a fear of the muscle stimulation as well as surprise at seeing one of their muscles contract without having intentionally contracted it themselves. A vasovagal reaction causes heart to slow down and blood pressure to drop, which produces a feeling weakness and a tendency towards fainting. If this does occur, all that is required is to stop the stimulation and for the person to lie down with the legs raised until the feeling of weakness disappears (5 to 10 minutes)
- Never allow muscular contraction during a stimulation session to result in movement. You should always stimulate isometrically; this means that the extremities of the limb in which a muscle is being stimulated must be firmly fixed, so as to prevent any movement that results from contraction.
- Do not disconnect any channel during the stimulation.
- Do not use the stimulator while driving or operating machinery.
- Do not apply stimulation during sleep.
- Do not use the stimulator at altitudes of over 3,000 meters.
- Always turn off the stimulator before moving or removing any electrodes during a session, to avoid electrical shock to the patient.
- Do not try to place electrodes on a body part not directly visible without assistance.
- Attach the electrodes in such a way that their entire surface is in contact with the skin.
- For obvious reasons of hygiene, each patient must have their own set of electrodes. Do not use the same electrodes on different patients.
- Some patients with very sensitive skin may experience redness under the electrodes after a session. Generally, this redness is completely harmless and usually disappears after 10 to 20 minutes. Never start another stimulation session in the same area, however, if the redness is still visible.
- Before each use clean and disinfect the motor point pen tip that is in contact with the skin.
- When using the device to customize programs, take special care that the parameters customized and applied by you to the patient are as you wanted them to be.

Equipment damage

- Check that the voltage and frequency ratings of your local **power line** are those indicated on the type plate of the power supply.
- Do not expose the Rehab/Theta/Physio/Wireless Professionalto direct sunlight, because some of the components may reach unacceptably high temperatures.

IMPORTANT INFORMATION - EN

- The unit is not protected from the ingress of water droplets from rain if used outside the carrying case
- The presence of children, pets and vermin does not normally affect the proper functioning. However, make sure that these sources do not contaminate the physiotherapy unit and keep them away from it. Also, keep the unit clean and protect it from dust and lint. The safety rules and regulations set forth apply in any case.
- It is recommended to use the transport bag that comes with the unit, for transport of the device, and to use a proper transport box to ship it.
- Always use the AC adaptor (power supply) provided by the manufacturer to recharge the unit.
- Do not store the device for a long time with empty batteries.
- Only use electrodes and motor point pen supplied by the manufacturer. Other electrodes and motor point pens may have electrical properties that are unsuitable for or may damage the Rehab/Theta/Physio.
- **Size of electrodes.** Do not use electrodes with an active area of less than 16 cm² due to the risk of associated burning. Proceed systematically with caution when the density of the current is over 2 mA/cm².
- Do not place the electrodes or pen in water.
- Do not apply solvents of any kind to the electrodes or pen.
- Skin irritation. Some people, with very sensitive skin, may experience redness under the electrodes after a session. Generally, this redness is totally harmless and usually disappears after 10 to 20 minutes. However, never start another stimulation session on the same area if the redness is still visible
- **Instructions for electrodes.** See the usage and storage instructions displayed on the bag of electrodes

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