READ THE MANUAL BEFORE OPERATING THIS PRODUCT

COMPEX ®

The logo symbol consists of a blue circle containing a white stylized geometric shape that resembles a cross or a square with rounded corners and a central square cutout.

ION

Vibrating Massage Roller

INTRODUCTION

The Complex ION helps to gain relief from sore or stiff muscles by working out deep layers of tissue. Made with industrial grade material this roller is heavy duty and will not lose its shape under any weight bearing circumstance. A polyurethane finish was used to make the outside feel like rubber. The textured finish allows for a solid grip on any area of the body you are trying to roll out. Featuring four vibration speeds, this roller offers a variety of vibration intensity to meet your recovery needs. Portable and easy to recharge with the micro USB charger.

- (A)** Speed level Indicator (Green)
- 4 levels: 1,2,3,4
- (B)** Power Switch
- (C)** Power Indicator (Blue) -
4 levels: 25%, 50%, 75%, 100%
- (D)** Micro USB charging port
(5V 2A)

BASIC SPECIFICATION		SPEED 1	SPEED 2	SPEED 3	SPEED 4	REMARK
Noise(dB)		<53db	-	-	54db	Condition: 30cm distance
Hertz (Hz)		31Hz	48Hz	53Hz	72Hz	
Motor Speed		1900	2900	3200	4320	
Power consumption testing	Voltage (V)	8.15	8.07	7.99	7.88	
	Current (A)	0.79	1.2	1.45	1.68	
	Power (W)	6.4W	9.6W	11.6W	13.2W	
Charge and Discharge test	Charge time	1.5H-2H				
	Discharge time	2-2.5H				Every 15 minutes of work automatically stops for one round
Adapter	US spec AC100-240V 50/60HZ, Output 5V 2A. 1.5m USB charging cable (Certification: uL/ETL ROHS)					
Battery	2*7.4V 2200mAh 18650 Battery 2 series 1 parallel					

FUNCTION INSTRUCTIONS

The product has four speed levels of different vibration intensity. To power on/off, press down the power button for 3 seconds until the power indicator light illuminate/turn-off.

SPEED 1: Power indicators on, and level 1 indicator (green) is on (1900 RPM)

SPEED 2: Two green lights on (2900 RPM)

SPEED 3: Three green lights on (3200 RPM)

SPEED 4: Four green lights on (4320 RPM)

4 power indicating lights illuminate along with speed level indicators, from 25%, 50%, 75% to 100% battery life.

CAUTION

- It is not recommended to deplete the battery less than 20% for a long period of time. Please fully charge the device to avoid discharge of the battery when the device is lower than 25% battery life.
- In each mode, the device will automatically turn off after 15 minutes of use.

SAFETY WARNING

THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE FOLLOWING INSTRUCTIONS:

- Used by and/or administered by adults only.
- Stop immediately if you feel any pain or discomfort.
- Avoid contact with head and/or other bony areas.
- Use on body parts for no more than 1 minute per specific location. Keep moving over muscles and avoid bony areas.
- Do not drop Ion™.

- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.
- The appliance is only to be used with the power supply unit<model ICP15A-050-2000B> provided with the appliance.
- **WARNING:** For the purposes of recharging the battery, only use the detachable supply unit provided with this appliance.

CONTRAINDICATIONS

PLEASE DO NOT USE ION™ WITHOUT RECEIVING APPROVAL FROM YOUR DOCTOR IF THE FOLLOWING CONDITIONS APPLY:

- Recent surgeries of any kind
- Skin conditions and/or wounds affected by vibrating massagers
- Herniated disks
- Pregnancy
- Diabetes/neuropathy or retinal damage
- Have a pacemaker
- Injuries affected by vibrating massagers
- Epilepsy or migraines
- Disc herniation
- Spondylolisthesis
- Vertebral detachment or spine joint rigidity
- Recent replacement of joints or IUD
- Metal pins or metal plates
- Any health-related diseases

WARNINGS

- Do not use if injured.
- Do not immerse in water, sand and/or loose debris that might get into motor housing vent.
- Do not remove screws or attempt to disassemble.
- This appliance contains batteries that are non-replaceable

ADVERSE REACTIONS

You may experience skin irritation and mild bruising if Ion™ is used in one spot for more than 60 seconds or over bony areas. Adverse reactions may be experienced by people recovering from surgery, skin conditions, wounds, nerve conditions, broken bones and/or contusions. To avoid reactions, users should consult doctor prior to using Ion™.

CHARGING INSTRUCTIONS

1. Please fully charge the device before first use.
2. While charging, 4 LED lights will flash. Based on the battery life level, the LED lights will turn on accordingly. When 4 LED lights turn on, the battery is fully charged and the adapter's indicator will turn green. Red indicator light is charging and green is fully charged.
3. The product cannot be used during charging
4. Please use the adapter provided. Do not use other adapters to avoid damage to the battery and potential danger.
5. Charging time: 1.5-2 hours.

DECLARATION OF CONFORMITY

FCC PART 15B Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

COMPEX VIBRATING MASSAGE ROLLER

MODEL: INTL-CX202WL02

Powered by Lithium-ion battery (7.4V , 2200mAh, 16.28Wh)

Rated Input: 5V 2A

Rated Power: Max. 13.2W

Use only with <ICP15A-050-2000B> supply unit

Made in China

WARRANTY

This device carries a limited warranty of two years from the date of delivery. The warranty applies to the device only, and the battery is not covered by this warranty.

During the warranty period, defective items will be repaired or replaced at no charge. Any evidence of misuse, abuse, alterations, or externally caused damage may have this warranty invalid.

For more information, please contact the distributor/manufacturer.

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www.compex.com

Copyright © 2020 by DJO, LLC

Individual results may vary. Neither DJO, LLC nor any of its subsidiaries dispense medical advice. The contents of this document do not constitute medical advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

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Copyright © 2020 par DJO, LLC
Copyright © 2020 de DJO, LLC
Distribuído por DJO France S.A.S