PUSH MED ANKLE BRACE AEQUI FLEX

INTRODUCTION

The Push med Ankle Brace Aequi Flex provides support in the treatment and prevention of ankle injuries. The brace provides improved stability which increases the physical strength. Mechanical support and adjustable pressure ensure the performance of the product.

Please read the product information carefully before using Push Aequi Flex. If you have any questions, please contact your supplier or send an email to info@push.eu

PRODUCT CHARACTERISTICS

Push Aequi Flex strikes a balance between good wearing comfort and a high level of mechanical support for the ankle. The function of the brace is formed by a Flex-protect reinforcement: a shell that is rigid at the core and flexible at the edges. It surrounds the inside of the lower leg, ankle and heel and has a cut-out for the malleolus. On the outside of the ankle, the brace consists of a functional strap that anchors the foot in the shell and moulds to the ankle when applied.

The Flex-protect system is fastened by attaching the diagonal strap at the front. This fastener consists of an elastic and a non-elastic part. The non-elastic part, together with the shell, prevents the risk of a sprain. The elastic fabric ensures a comfortable fit and enclosure of the joint. The application is completed by wrapping the two elastic straps, with which the desired pressure is applied around the ankle using the selected tension.

Push Aequi Flex can be used for the functional treatment of ankle injuries and for long-term use in cases of chronic instability. Functional treatment means that putting weight on the foot and ankle with the brace, as prescribed by the doctor, is part of the treatment. The outward and inward tilting of the foot is restricted, while sufficient freedom is allowed for bending and stretching.

The numbering on the straps makes the brace easy to apply. Push Aequi Flex is worn over the sock. Optimal stabilisation is achieved when the brace is worn in a closed (face-up) shoe.

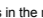
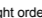
INDICATIONS

- Acute lateral ankle sprain
- Follow-up treatment of ankle fractures treated conservatively or surgically
- Moderate to severe instability
- Moderate to severe osteoarthritis
- Secondary prevention of ankle sprain

There are no known conditions where the brace cannot or should not be worn. In the case of circulatory problems a brace may cause congestion. Do not apply the brace too tightly at first. In diabetics in particular, make sure that no folds or other pressure points develop after application.

Pain is always a warning sign. In the case of persistent pain or worsening of the symptoms, it is recommendable to consult your attending physician or therapist.

FITTING

The hook and loop fasteners are marked with symbols. These help you to close the fasteners in the right order. First, the fastener with  on , then , etc.

- Push Aequi Flex is worn over the sock and in the shoe.
- Place the brace around the foot. The reinforcement is on the inside of the foot. Place the long straps to the sides (figure A).
- Fasten the diagonal strap, marked with this symbol:  (figure B).
- Take the elastic strap on the outside of the foot, marked with this symbol . Pull it diagonally across the instep, wrap it around the lower leg just above the heel and fasten it with the hook and loop fastener (figure C).
- Finally, take the second elastic strap on the inside of the foot (figure D). Pull it diagonally across the instep, wrap it around the lower leg slightly higher than the first strap and fasten it (figure D).

WASHING INSTRUCTIONS

To promote hygiene and to maximise the lifespan of the brace, it is important to wash the brace regularly, following the washing instructions. If you are not going to use the brace for a longer period of time, it is advisable to wash and dry it before storage. Before washing, close the hook and loop straps to protect the brace and other laundry. The brace can be washed at up to 30 °C in a delicate wash or by hand. Do not use any bleaching agents. The brace may be lightly spun, then hang the brace up to dry in the open air (not near a heater or in a tumble dryer).

IMPORTANT INFORMATION

- Correct use of your Push brace may require a diagnosis by a physician or therapist.
- Optimal stabilisation is achieved when the brace is worn in a closed (face-up) shoe. Be careful not to slip if the brace is worn without a shoe.
- Push braces were designed to be worn on an undamaged skin. If the skin is damaged, it must first be covered with appropriate material.
- The correct choice of size is essential for optimal functioning. Try on the brace before purchasing to check its size. Your retailer's professional staff will explain the best way to fit your Push brace. Always keep the product information so that you can read over the fitting instructions later.
- Consult a brace specialist if the product does not fit well.
- Check your Push brace before every use for signs of wear or ageing on any parts or seams. Optimal function of your Push brace is only guaranteed if the product is completely intact.
- For the durability of the brace, it is important that the brace is not exposed to temperatures above 70 °C.
- Using the brace in water containing salt or chlorine may result in a reduced lifespan.
- In accordance with EU regulation 2017/745 on medical devices, the user must report any serious incident related to the product to the manufacturer and the competent authority in the EU Member State in which the user is based.

Push® is a registered trademark of Nea International bv
Patent pending.

For details of the distributor in your country, refer to www.push.eu/distributors

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