Quick Start for TENS/EMS Programs

Before beginning any therapy, we recommend you talk to your healthcare provider and read the full user manual.


- 1 Insert batteries provided into back of unit
- 2 Attach lead wires to bottom of unit
- 3 Leaving electrodes on the plastic backing, attach lead wires to electrodes

TENS therapy is used for temporary and long term pain relief. Pages 26 - 37 show electrode placement and program numbers

EMS can reduce muscle spasms, strengthen muscles, increase blood circulation and reduce edema. Pages 38 - 47 show electrode placement and program numbers

- 4 Carefully remove electrode shields minimizing contact to your fingers and apply the electrodes as shown.
- 5 Turn your StimTec PLUS on by pressing the button on top of the unit
- 6 Switch between TENS and EMS modes by holding the PROG button for 3 - 4 seconds.
- 7 Press the PROG button quickly until the display shows your program number and body targets
- 8  For **EMS** programs, select a FORCE using the double arrow button
- 9  Using the arrow buttons, increase the intensity of each channel. Choose the highest intensity that is still comfortable.

NOTE All other settings are pre-selected by the specific program and do not require changes.

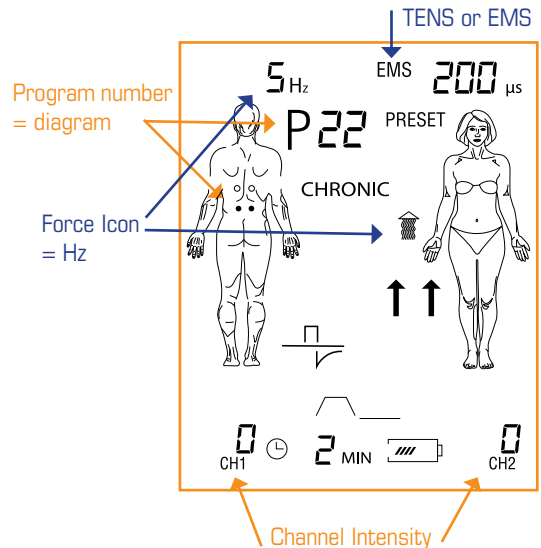EMS ONLY Increasing Hz, increases force of the muscle contraction. Choose the highest force that is still comfortable.

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