


SaeboStep Barefoot Accessory Strap




SaeboStep sold separately.

PARTS OF THE DEVICE

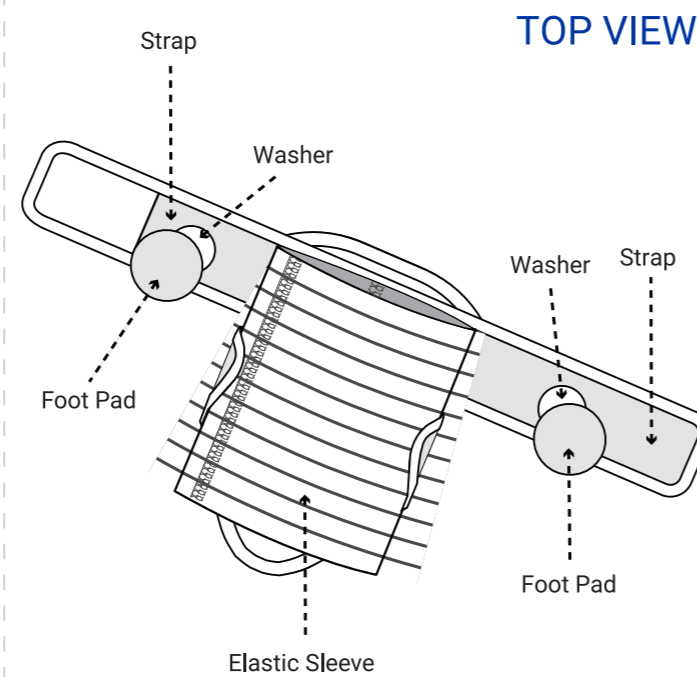

TOP VIEW

PARTS OF THE DEVICE

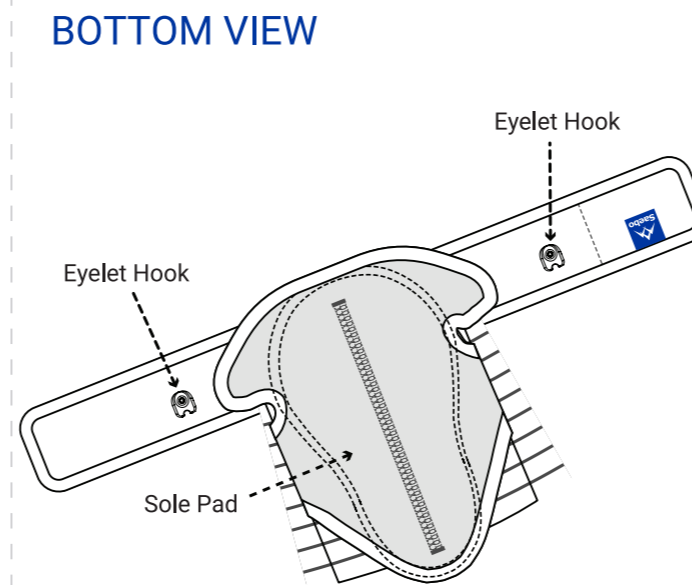


BOTTOM VIEW

1

APPLY SAEBO STEP

Apply Saebo Step to ankle per instructions.

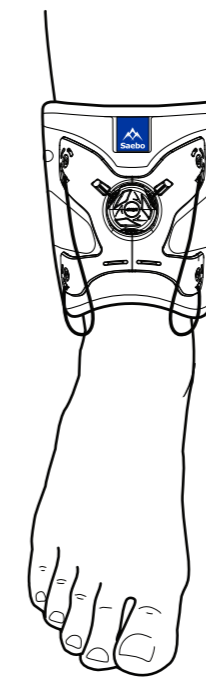


2

APPLY SAEBO BAREFOOT


Slide Barefoot Accessory Strap to just past toes as shown.

Tip: If your toes curl or flex during walking, consider positioning the strap down to block toes from curling.

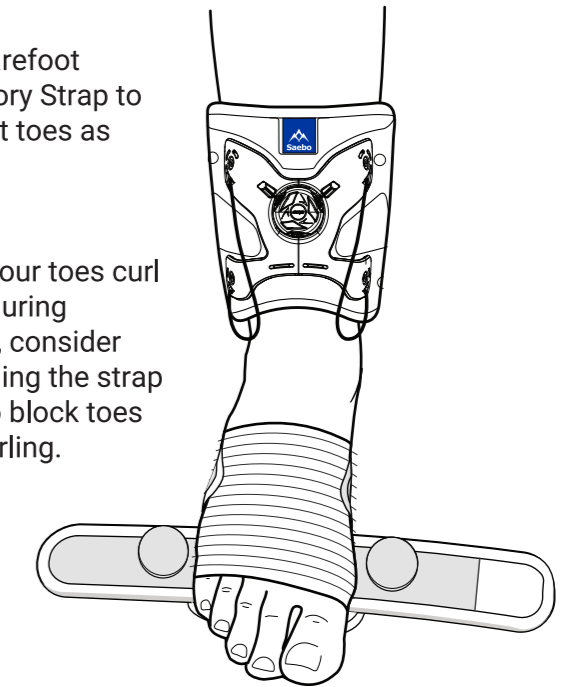

3


SECURE STRAPS

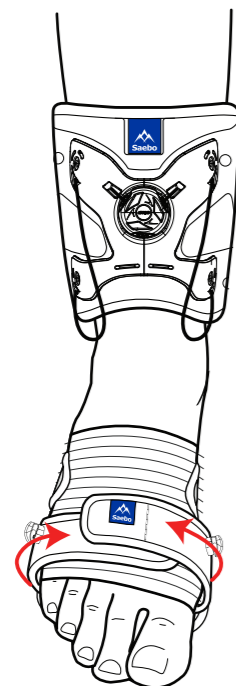

Secure straps across foot.

4


APPLY CORDS TO HOOKS

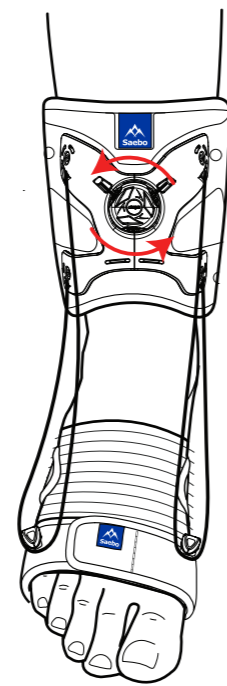

Turn Saebo Step dial counterclockwise to loosen cords. Pull left and right cords at the same time and loop onto hooks.

5

ADJUST POSITION

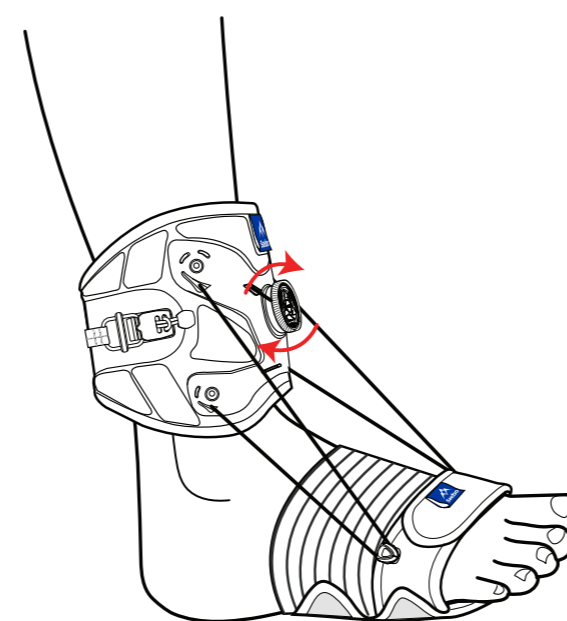

Tighten cords by turning dial clockwise to raise foot to desired position.

SEE THE VIDEOS

Visit our website to see instructional videos including how to replace eyelet hook hardware.

www.saebo.com/saebobarefoot



Saebo, Inc.

2459 Wilkinson Blvd., Suite 120-B
Charlotte, NC 28208

888-942-5751
sales@saebo.com
www.saebo.com



European Healthcare & Device Solutions (Ireland) Ltd.
info@europeandevicesolutions.co.uk
www.europeandevicesolutions.co.uk

Australian Sponsor:
Ausmedic Australia Pty Ltd
Unit 3, 3 Basalt Road Pemulwuy NSW
2145 1300 473 422
Performhealth.com.au

