

UNLOADER ONE® X

Unloader One X improves mobility by providing pain relief from knee osteoarthritis and degenerative meniscal tears.

It features Össur's unique patented and clinically proven 3-point Leverage System that generates unicompartmental unloading of the knee joint.

INDICATIONS FOR USE

- Moderate to severe unicompartmental knee osteoarthritis
- Degenerative meniscal tears
- Other unicompartmental knee conditions that may benefit from unloading such as:
 - Articular cartilage defect repair
 - Meniscal repair
 - Avascular necrosis
 - Condylar bone marrow lesions
 - Tibial plateau fractures




Watch the Unloader One X Fitting Video




FEATURES AND BENEFITS

- 3-Point Leverage System consisting of two Dynamic Force System (DFS) Straps, rigid shells and a single upright hinge contribute to a unicompartmental unloading of the knee joint.
- The unloading is effective only when needed, no condyle pressure during sitting or swing phase.
- A slender perforated frame with soft edges, contours easily to individual anatomies and remains discreet whether worn over or under clothing.
- Skin friendly liners and pads as well as a smooth inner surface of shells reduce probability of pressure points.
- The gastroc strap is positioned over thickest part of calf to help prevent migration.
- Tool-free SmartDosing® dials enable the patient to be in control of their pain level by fine-tuning the strap tension.
- Color-coded touchpoints and Quick-Fit buckles increase user-friendliness.
- Alignment guide on hinge facilitates correct positioning on the leg.
- Entire brace is machine washable (washbag included with product).
- Easily adaptable to individual needs and anatomical shapes.
- Field serviceable.

REQUIRED MEASUREMENTS

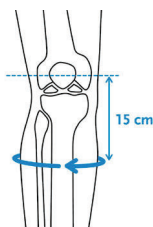

Measure calf circumference 15 cm (6") below mid-patella. Leg in extension.

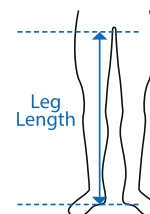


Size	Calf circumference 15 cm (6") below mid patella	
	cm	inch
XS	25 - 31	9¾ - 12¼
S	31 - 36	12¼ - 14
M	36 - 43	14 - 16¾
L	43 - 51	16¾ - 20
XL	51 - 62	20 - 24¾

UNLOADER ONE® X

ORDERING INFORMATION

STANDARD


Size	Right Medial	Left Medial	Right Lateral	Left Lateral
XS	UOX0518711	UOX0618711	UOX0528711	UOX0628711
S	UOX0518712	UOX0618712	UOX0528712	UOX0628712
M	UOX0518713	UOX0618713	UOX0528713	UOX0628713
L	UOX0518714	UOX0618714	UOX0528714	UOX0628714
XL	UOX0518715	UOX0618715	UOX0528715	UOX0628715

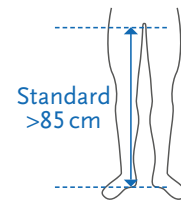

Standard is recommended for patients with a leg length greater than 85 cm (33.5").
Hinge is placed on the side of the affected compartment

SHORT

Size	Right Medial	Left Medial	Right Lateral	Left Lateral
XS	UOX1518711	UOX1618711	UOX1528711	UOX1628711
S	UOX1518712	UOX1618712	UOX1528712	UOX1628712
M	UOX1518713	UOX1618713	UOX1528713	UOX1628713
L	UOX1518714	UOX1618714	UOX1528714	UOX1628714
XL	UOX1518715	UOX1618715	UOX1528715	UOX1628715

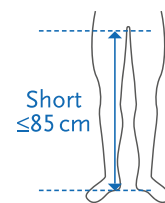

Short is recommended for patients with a leg length less than or equal to 85 cm (33.5").
Hinge is placed on the side of the affected compartment.

ACCESSORIES AND REPLACEMENT PARTS

THIGH LINERS

Description	Thigh Right Medial, Left Lateral	Thigh Right Lateral, Left Medial	Thigh Extension Right Medial, Left Lateral	Thigh Extension Right Lateral, Left Medial
Gray, Sensil	UOX526201	UOX525201	UOX546201	UOX545201
Gray, ActiveGrip	UOX526301	UOX525301	UOX546301	UOX545301
Gray, Doeskin	UOX526101	UOX525101	UOX546101	UOX545101

CALF LINERS

Size	Right Medial, Left Lateral	Right Lateral, Left Medial
Gray, Sensil XS-M	UOX516152	UOX515152
Gray, Sensil L/XL	UOX516162	UOX515162
Gray, ActiveGrip XS-M	UOX516153	UOX515153
Gray, ActiveGrip L/XL	UOX516163	UOX515163
Gray, Doeskin XS-M	UOX516151	UOX515151
Gray, Doeskin L/XL	UOX516161	UOX515161

THIGH AND LOWER DFS STRAP*

	Right Medial, Left Lateral	Right Lateral, Left Medial
*Includes Thigh and DFS Strap with buckle	UOX406102	UOX405102

CALF AND UPPER DFS STRAP*

	Right Medial, Left Lateral	Right Lateral, Left Medial
*Includes Calf and DFS Strap with buckle	UOX406101	UOX405101

ANTI-MIGRATION PILLAR

Right Medial, Left Lateral	Right Lateral, Left Medial
UOX700025	UOX700045

UNLOADER ONE® X

OTHER ACCESSORIES AND REPLACEMENT PARTS

Description	Part#
Gray, Alligator Kit*	UOX900002
Gray, Hinge Pad	UOX700011
Gray, Popliteal Pad	UOX700092
Gray, Popliteal Pad - Short	UOX700082
Gray, Sensil Strap Pad	UON0182
Gray, Sensil Strap Pad - 10 Pieces	UOX700193
Gray, Smartdosing Dial	UOX1000002
Gray, Strap Wrap Set	UOX700094
Ease of Use Kit**	UOX800001
Flexion/Extension Stop Set	B-301040047
Sundial	UOX1200001
Wash Bag	UON60200

*Includes 4 Alligator Clips: 2 DFS, 1 Thigh and 1 Calf

**Includes Sundial and 4 buckle finger-loops

SLEEVE

Size	Part# - Black - Sleeve with Mesh*	Part# - Black - Össur**
XS	SLV60011	B-250150101
S	SLV60012	B-250150102
M	SLV60013	B-250150103
L	SLV60014	B-250150104
XL	SLV60015	B-250150105
XXL	SLV60016	B-250150106

*Sleeve with Mesh - Main fabric: 92% Cotton, 8% Spandex - Mesh:100% Polyester - Band: 80% Polyester, 20% Spandex

**Össur Sleeve - 80% Nylon, 20% Spandex