




Spine Bracing System

PATIENT APPLICATION INSTRUCTIONS

EXOS FORM™ II 631 / 637

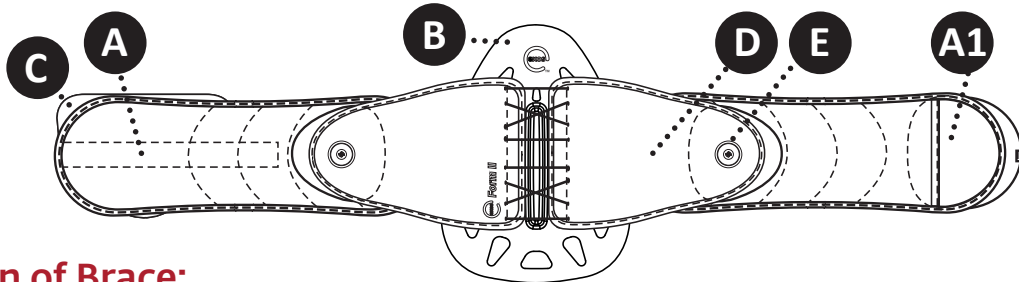

Application of Brace:

1 LOOSEN BRACE by pulling out on both Boa Knobs (E) and pull Rear Panel Section (D) in opposite directions.



2a APPLY BRACE by inserting hand into the Donning Mitt pocket (A1) and wrapping the Belt Wings (A) around the waist.



2b NOTE: Center Tall Rear Panel (B) on back. The Exos logo on the Front Panel (C) should be facing upright and away. The point of the Tall Rear Panel should be facing up.



3 TIGHTEN BRACE by pushing in both Boa Knobs and turn knobs clockwise.



NOTE: Left Boa Knob controls top half of brace. Right Boa Knob controls bottom half.

OPTIMAL COMPRESSION: When sitting down, prior to sitting, release compression by pulling-out on Boa Knobs. Once seated, tighten compression by pushing in on Boa Knobs. When standing up, tighten compression if necessary.

BRACE REMOVAL: Loosen brace compression by pulling-out on Boa Knobs. Detach hook and loop connection on Belt Wings and remove brace.

USE AND CARE (CLEANING & STORAGE INSTRUCTIONS): Hand wash in cold water, less than 86°F (30°C), using mild soap. Air dry. **NOTE:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material. Do NOT iron. Do NOT expose to heat over 120°F (49°C). Do NOT tumble dry. Do NOT bleach.

MATERIAL CONTENTS: Nylon, Polyethylene, Polyester, Antimicrobial, Polyurethane, Stainless Steel, Polycarbonate, Thermoplastic Elastomer, Polyoxymethylene, Carbon Steel, Acrylic Adhesive. **FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.**

CONTRAINDICATIONS: Pregnancy and circulation, pulmonary, cardiovascular or skeletal conditions which have risk to be made worse as a result of compression and/or pressure.



For more information including application videos, please visit

DJOGlobal.com/products/exos/exos-form-ii-back-brace or call **888.405.3251**

For product assistance, please contact Product Support at **1.888.405.3251** or email product.specialist@djoglobal.com

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

DJO, LLC | A DJO Global Company
T 800.336.6569 **D** 760.727.1280 **F** 800.936.6569
 1430 Decision Street | Vista, CA 92081-8553 | U.S.A.
DJOGlobal.com/exos

U.S.A. and International Patents Pending ©2016
 DJO, LLC 00-4189 Rev A

Next Appointment:

Date: _____

Time: _____



MOTION IS MEDICINE™